LEES DIT EERST!

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READ THIS FIRST!

- **Important!** The cotton swab should be used for only one of these locations.

Your doctor has determined the location (throat, vagina or rectum) for your culture. If you are on your period, wait to collect a specimen until your period is over.


Step 1:

- First open the package, then wash your hands with water and soap.
- Remove the cotton swab and the test tube. Hold the cotton swab in the middle between your thumb and forefinger. **NOTE; don't touch the cotton tip!**

Step 2a:
Taking a Throat Swab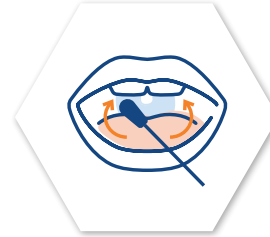

- Insert the cotton swab as deeply as possible into the back of the throat and rotate it while running it from left to right over the back wall of the throat.
- **Go on to step 3.**

Step 2b:
Taking a Vaginal Culture

- Get into a comfortable crouching position and insert the cotton swab about 5 cm into the vagina.
- Turn the swab around at least 3 times for about 10 seconds, making sure touching the vaginal wall.
- **Go on to step 3.**

Step 2c:
Taking a Anal Culture

- Insert the cotton swab cautiously into the anus (like a fever thermometer).
- Take the end of the cotton swab between thumb and index finger, rotate it and then withdraw it from the anus.
- **Go on to step 3.**

Step 3:
Collection

- Unscrew the cap of the test tube that is provided (**BE CAREFUL: there is liquid inside**).
- Put the cotton swab into the test tube. Align the stripe on the shaft (break line) with the upper edge.

- Put the shaft against the edge of the test tube. Bend the part that is sticking out and break off the shaft.
- Throw away the end piece. Firmly tighten the cap.

Step 4:
Identification of the test tube

- Stick the provided barcode sticker lengthwise onto the test tube.
- Put the test tube in de plastic blister en close the blister.


Step 5:
Preparation for shipping

- Fill in your personal data, the collection date and **where (throat, vagina or anus) the culture was taken from**, on the included form.

- Put the blister with the test tube with the form into the plastic bag.
- Remove the backing from the adhesive strip of the plastic bag and seal it shut.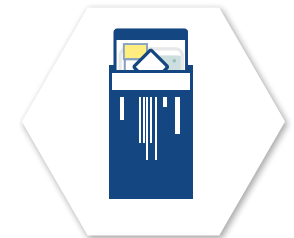

- Hand in the bag at your general practitioner, drop of at a postNL-point, or in an orange mailbox. Check: postnl.nl/locatiewijzer/, check; Brief posten. On sundays and holidays also check; Medisch.